CE 1639

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EN Instructions for use

Description
Biolight Plus is assembly device of radiopaque, light-conducting and small diameter dental post, made of a glass fiber-reinforced composite. Posts are intended for the continuous reinforcement of the root canal and the coronal part. The use of an adhesive technique to bind together the post, the core material and the root dentin shows long term favorable outcome.

The polymer matrix is composed of urethane dimethacrylate (UDMA) and doesn't contain BisGMA.

Indication

Endodontically treated teeth post and core build-up restorations. Various configurations are available to cover all practitioners' needs (the appropriate size will be chosen by means of a radiographic image).

Spatial adaptability exempts the operator of any boring.

Contra-indications
- Allergy to methacrylates.
- Less than 2-3 mm supergingival tooth structure.

Side effects
At the present state of knowledge, no side effect has been reported.

Precautions

- Dental posts must be cleaned only with medical alcohol prior to insertion in the canal.
- Avoid direct contact with fingers.
- The coronal adjustment should be done under water spray and a surgical suction device should be used. We recommend the use of a dental dam. Protective glasses, surgical mask and gloves should be worn during any milling.
- In case of irritation the discomfort can be eased with water and mild soap (mechanical action).
- Dental post is a single-use device and must not be reused.

- Biolight Plus instructions for use**
- Remove gutta percha from the canal. Leave a 5 mm apical seal.
 - Choose the Biolight Plus size and number according to the morphology of canal.
 - Clean posts with alcohol and control that the position in the mouth is correct.
 - X-rays control.
 - Clean and dry the canal with air and paper points (remains of eugenol can inhibit the polymerization of most bonding materials).
 - The post must be only cleaned with alcohol, before definitive implementation.
 - Post bonding using an adhesive process :
 - Tooth preparation:
 - Etch the walls of the canal (37% phosphoric acid) and rinse thoroughly with water. Carefully dry using paper cones.
 - Use an adhesive for composite materials. We advise not to use self-etching acid adhesive primers (acid may inhibit the polymerization of the bonding material).
 - Introduce adhesive in the canal, evaporate and light cure.
 - Biolight Plus preparation:
 - Coat the post with a silane for composites then let it dry.
 - Use an adhesion primer for composite materials
 - Fill uniformly each post with adhesion primer, evaporate the excess with a dried air. Light cure quickly.

-Assembly and insertion:

- We advise to use a dual curable bonding for composite posts or a self-curable bonding material (to the exclusion of the 100% photo bonding material).
- Away from light, paint each post with bonding agent using a brush.
- Fill the canal with bonding agent using a root canal filler or micro-syringe.
- Insert the number of bundles in accordance with determined sequence during fitting.
- Maintain pressure. Cut the coronal extend of the post using only thin scissors.
- Each post is independent in the canal, arrange it homogeneously in the bonding agent favoring canal rim.

- Light cure or allow the cement to set undisturbed for 10 minutes.
- Core build-up :
 - Process according to the professional practice for composite filling materials.

Cleaning and disinfection (Products are not delivered sterile)

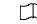
- The post can be disinfected with medical alcohol.

Guarantee - Limited liability
The guarantee is limited to the quality of the product: any defective product will be replaced.
The Batch Number must be cited in all correspondence which ask for identification of the product.
This product has been developed for use in dentistry and must always be used according to the Directions. Any harm resulting from a failure to comply with these provisions, incorrect handling or utilisation for purposes other than those indicated in the Directions, will not be the manufacturer's responsibility. Before every use, the user is obliged to check the compatibility of the equipment with the intended application; in consequence, he or she is entirely liable for use of the product and any associated damages.

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Products Biolight Plus are made in France.

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